

User instructions

Home Chemotherapy Spill Kit

Contents

- Instructions for use
- 1 pr Nitrile Gloves
- Goggles
- 1 x 5 Dry Wipes
- 1 Alcohol Wipe
- 1 Resealable Bag
- 1 Chemo-Waste Bag
- Report Form

Preparation

- + Read all the instructions before proceeding, taking time to identify the components.
- + Evacuate everyone away from the area.
- + Lay out all the components of the spill kit.
- + Put on the gloves first.

Important

- + Do not touch your face, clothing or any other area, item or person until safe disposal of all spilled hazardous materials.

Cleaning

- + Use the dry wipes to collect any damaged bottles, vials or syringes and place into the clear re-sealable bag. (Needles should be capped before placing into bag or placed into a proper sharps waste bin if available).
- + Use the dry wipes to mop up any spill, working away from the body in a single wiping motion. Do not re-wipe over an area with the dirty wipe – always use a clean wipe.
- + Place dirty wipes into the re-sealable bag.
- + After dry wiping, the area should be cleaned using the alcohol wipe. Place the soiled wipe into the re-sealable bag.
- + Place any soiled items (eg. Clothing) into Chemo-Waste Bag.
- + Remove protective glasses and place into re-sealable bag.
- + Carefully remove gloves and drop into the re-sealable bag.
- + Close the re-sealable bag and place into the Chemo-Waste Bag.
- + Peel tamper evident closure and seal the bag.
- + As a precaution, wash hands thoroughly and consult a doctor if any skin contact with cytotoxic drug has occurred.
- + Complete enclosed Report Form.
- + Contact the issuing hospital or doctor for instruction on how to return to them, the report form and the sealed Chemo-Waste Bag for proper hazardous waste disposal (do not dispose of in domestic waste).

Order Code CS0999

HELAPET

Circle Business Centre
Blackburn Road
Houghton Regis
Beds. LU5 5DD
0800 0328 428
www.helapet.co.uk

**Disposables for
Hospital Pharmacy**