

## MicroTak Mats

### Clean Rooms




- Reduces dirt and particles entering the clean room
- Easy Peel – Disposable
- No Washing
- Anti-microbial properties

### Theatres

Main \* Gynaecology \* Day

- Low cost disposable mat
- Effective removal of soiling from shoes and trollies
- MicroTak Mat inhibits microbial growth

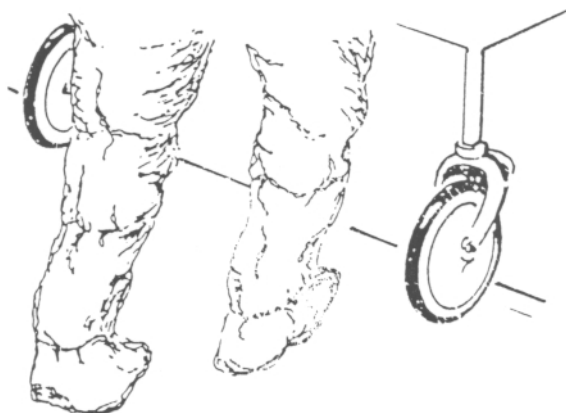

Freefone Orders: 0800 0 328 428

**Helapet Limited**  
Circle Business Centre  
Blackburn Road  
Houghton Regis  
Bedfordshire  
LU5 5DD  
England

Tel: 01582 501980  
Fax: 01582 501981  
[www.helapet.co.uk](http://www.helapet.co.uk)  
E-mail: [info@helapet.co.uk](mailto:info@helapet.co.uk)

A CliniMed® Group Company



CERTIFICATE No. FS 52480  
BS EN ISO 9002  
BS EN ISO 13485

# MicroTak Mat

## Technical Information

### Construction

Integral corner tabs, numbered consecutively

Standard Packs: 120 shts/pk made up of : 4 mats of 30 shts/pk or 6 mats of 20 shts/pk.

Speciality Packs: Manufactured to customer specifications.

Floor Attachment options:

Full bottom sheet adhesive coated or  
2 inch strips of double faced tape.

Sizes:

Stock	18 x 45”(MF1845)	26 x 45”(MF2602)	
Standard	18 x 36” 36 x 45”	24 x 36” 36 x 60”	
Non-Standard	Any size up to 45” wide x 130” long.		

Colours:                      White                      Grey                      Blue                      Clear

### Installation and Use

1. Remove the protective covering from the adhesive bottom sheet and lay the mat on a clean floor or in a frame. Step on the edges of the mat to positively secure it to the floor.
2. Remove the protective film from the top of the mat.
3. The mat is now ready for use. When the exposed sheet is contaminated, peel it off and discard it, exposing a new clean sheet for use.

### Anti-Microbial Properties

All **MicroTak Mats** have anti-microbial adhesive to control microbial growth on the mat. The active ingredients are 5-Chloro-2 methyl-4-isothiazolin, and 2-Methyl-4-isothiazolin-3-one. The microbicide is manufactured by Rohm & Haas Corporation and is sold under the trade name Kathon LX 1.5%

MicroTak Mats effectively resist Fungi, Gram negative and Gram positive bacteria. Full data available on request.

Product	Size	Code	Qty
Grey MicroTak Mat	18” x 45”	MF0004	4 x 30 shts/case
White MicroTak Mat	18” x 45”	MF1845	4 x 30 shts/case
Blue MicroTak Mat	26” x 45”	MF2602	4 x 30 shts/case
Grey Hospital MicroTak Mat	18” x 45”	MH1845	6 x 30 shts/case
White MicroTak Mat	18” x 36”	MF1836	4 x 30 shts/case
White MicroTak Mat	24” x 36”	MF2436	4 x 30 shts/case